# PARAFINA BLANCA

Con Vitamina E

*mebaline*

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White Paraffin. Moisturizing. For professional use.

Paraffin should be applied by a professional and the intended use is warming the body part to be treated. The product should be heated in specific devices and it will be applied according to the professional. INSTRUCTIONS FOR USE: Use only in devices which are specific for paraffin fusion. Apply the product with a brush or by immersion, in case of feet and hands. The product should be applied at the minimum temperature allowed by the device, without letting solidify the product. PRECAUTIONS: Not recommended for children under 3 years. External use. Do not apply on eyes, mucous membranes, scalp or damaged skin. Pay special attention to the temperature while applying the product, to avoid risk of skin injuries or burns. Store in a fresh place.

INGREDIENTS: FULLY REFINED PARAFFIN WAX (PARAFFIN WAXES HIDROTREATED, SLACK WAXES HIDROTREATED), PARAFFINUM LIQUIDUM, CERA MICROCRISTALINA, TOCOPHERYL ACETATE.

Fabricado por:  
Laboratorios Ismar, S.L.  
Pl. Salelles, C/Mestre Cabanes,7  
08253 Sant Salvador de Guardiola (Spain)  
Distribuido por:  
Bastos Medical, S.L.

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Mint Paraffin. Moisturizing. For professional use.

Paraffin should be applied by a professional and the intended use is warming the body part to be treated. The product should be heated in specific devices and it will be applied according to the professional.

INSTRUCTIONS FOR USE: Use only in devices which are specific for paraffin fusion. Apply the product with a brush or by immersion, in case of feet and hands. The product should be applied at the minimum temperature allowed by the device, without letting solidify the product. PRECAUTIONS: Not recommended for children under 3 years. External use. Do not apply on eyes, mucous membranes, scalp or damaged skin. Pay special attention to the temperature while applying the product, to avoid risk of skin injuries or burns. Store in a fresh place.

INGREDIENTS: FULLY REFINED PARAFFIN WAX (PARAFFIN WAXES HIDROTREATED, SLACK WAXES HIDROTREATED), PARAFFINUM LIQUIDUM, CERA MICROCRISTALINA, TOCOPHERYL ACETATE, PARFUM, C.I. 19140, 42090.

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